


ViriMASK™

Protective Ocular-Respirator

FACE MASK WITH VISOR

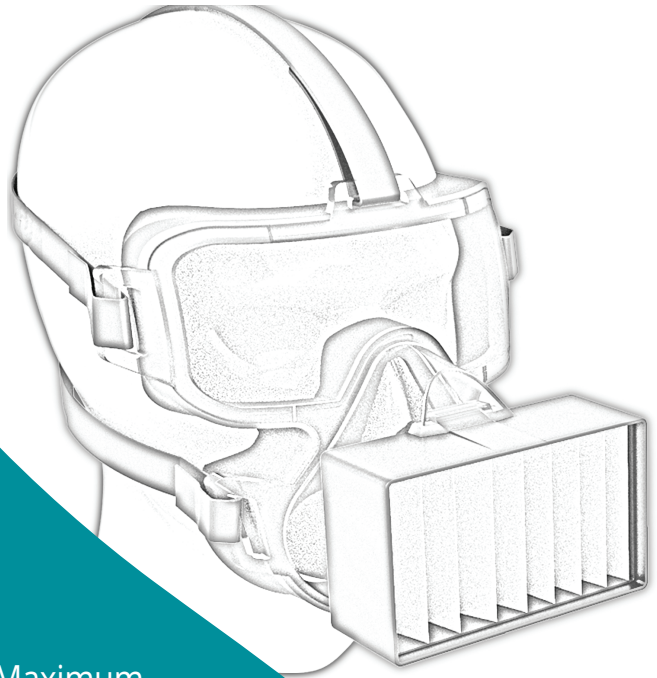

Protects

ViriMASK™ Combines Maximum Protection & Maximum Comfort against:

- Micron-size particles
- Bacteria & viruses including COVID-19 (Coronavirus)
- Exhaled aerosols
- Environmental aerosols

www.ViriMASK.com

ABOUT ViriMASK™

When ViriMASK™ is used correctly, the wearer will feel no noticeable change in breathing ability nor in voice volume.


ADVANTAGES

- Superb Seal: Completely sealed to outside air penetration
- Higher Filtration Rating: Protects against 99.99% particles
- Integrated Eye Protection: Prevents infection/contamination through the eye
- Exceptional Ventilation: Large breathing area
- Superb Personal Comfort: Minimal skin pressure for extended wear
- Reusable Mask: Can be washed and disinfected. Easy filter replacement, packaged for safe disposal



ViriMASK™

ViriMASK™ SIZING

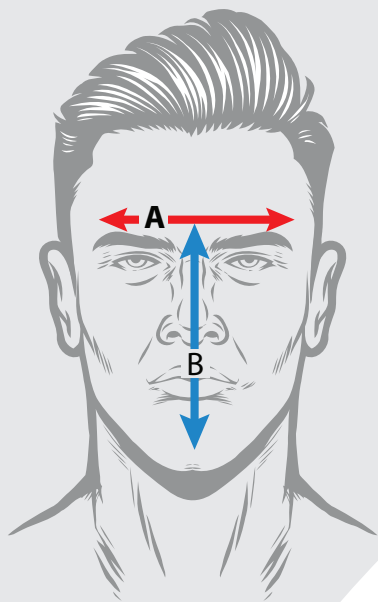

1. Measure the width of the forehead just above the supraorbital ridge (above the eyebrows). (A)
2. Measure the length of the face from line (A) above the eyebrows to the point just above chin (gingival line). (B)
 - If A is below 145 mm **OR** B is below 150 mm, use Medium.
 - If A is above 145 mm **AND** B is above 150 mm, use Large.
 - If A is below 130 mm **OR** B is below 130 mm, contact support (support@virimask.com) for advice.

COMPONENTS		
Catalog Number	Color	Product
VM-MASK-L-RD	Red	Large ViriMASK™ Kit (Mask Frame and Visor, 2 ViriMASK™ V60 Filters , and Head Straps)
VM-MASK-L-BL	Blue	
VM-MASK-L-BG	Beige	
VM-MASK-M-RD	Red	Medium ViriMASK™ Kit (Mask Frame and Visor, 2 ViriMASK™ V60 Filters , and Head Straps)
VM-MASK-M-BL	Blue	
VM-MASK-M-BG	Beige	
VM-Filter_5-RD	Red	Set of 5 ViriMASK™ V60 Filters
VM-Filter_5-BL	Blue	
VM-Filter_5-BG	Beige	
VM-RM-100-RD	Red	Replacement Head Straps
VM-RM-101-BL	Blue	
VM-RM-102-BG	Beige	

Applicable Standards: ASTM F2100 Level 3 | EN 14683:2019 Type IIR Barrier Protection
Before using the **ViriMASK™**, please read this *Instructions for Use Manual*.

Manufacturer:
ViriMASK Ltd. by Oneg HaKarmel Ltd.
4 Etgar St., Tirat Carmel 3910302, Israel
Tel. +972-4-8242369


US CONTACT

OHK Medical Devices, Inc.
2885 Sanford Ave. SW #14751, Grandville, MI 49418
Tel: +1.866.503.1470

SUPPORT@ViriMASK.com | www.ViriMASK.com

EU REPRESENTATIVE

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MedNet ECREP GmbH
Borkstraße 10, Münster 48163, Germany
www.medneteurope.com
ecrep@medneteurope.com



ViriMASK™

ViriMASK™ is a Trademark of Virimask Ltd., Tirat Carmel, Israel
© Copyright 2020 ViriMASK Ltd. All rights reserved.