


## ViriMASK™ SIZING

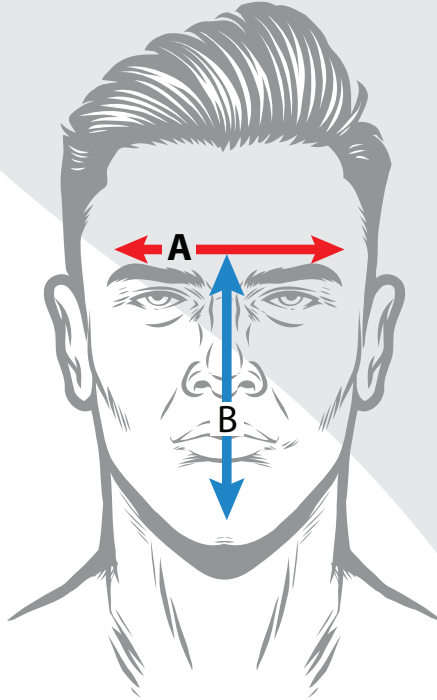

1. Measure the width of the forehead just above the supraorbital ridge (above the eyebrows). (A)
2. Measure the length of the face from line (A) above the eyebrows to the point just above chin (gingival line). (B)
  - If A is below 145 mm **OR** B is below 150 mm, use Medium.
  - If A is above 145 mm **AND** B is above 150 mm, use Large.
  - If A is below 130 mm **OR** B is below 130 mm, contact support ([support@virimask.com](mailto:support@virimask.com)) for advice.